



ANCHOR INSTALLATION INSTRUCTIONS

A. New Deck Construction

Follow recommended instructions D-1 thru D-6.

B. Existing Deck Construction

Any retro-fit anchor must be located at least 6" (152mm) from any drain, expansion joint or crack. Follow installation instructions D-1 thru D-6.

C. Alternate Core Drilling Installation

If a deck is 8" (203mm) thick or more, a core drilling method may be used. The core drill must cut at least a 4" (102mm) diameter hole. Follow installation instructions D-1 thru D-6.

D. Installation Instructions

1. Using the ADA guidelines, find the anchor location site.
2. Refer to Installation drawings.
3. Attach bonding wire to common bonding grid. Refer to NEC Section 680-22 for pools/spas. Use a minimum #8 A.W.G. solid copper wire.
4. Fill the anchor with newspaper to ensure interior is free of concrete when pouring.
5. Make sure anchor is level and finished top is between 1/8" and 1/4" below the finished deck level to insure proper anchor cap placement.
6. After the deck is finished, allow at least seven (7) days for it to cure before the lift is used.

